



Generation II Entree Bath – All Models

OPERATION MANUAL

ENTREE BATH TRAINING CHECK LIST

After reviewing the operation and safety instructions in this manual, the Entree Bath can be operated and all components can be identified.

OPERATION:	YES	NO
Turn On BathAire ^(SM) Control		
Move Individual Into The Entree Bath		
Secure Individual		
Close Tub Door With Push Buttons		
Close Drain With Open/Close Lever		
Adjust Mixing Valve To Desired Temperature		
Fill Tub with Tub Fill Valve		
Use Liquids As Desired		
Use Shower Wand With Shower Wand Control Valve		
Open Drain With Open/Close Lever		
After Water Is Drained, Open Tub Door With Push Buttons		
Move Individual Out Of The Entree Bath		
Return Individual To Room Or Desired Location		
Remove Comfort Pads And Place In Foot Well		
Use Disinfectant To Clean Tub And Pads		
Rinse Disinfectant From The Tub And Pads		

LOCATE & IDENTIFY:		
LCD Thermometer		
GFCI reset		
Aromatherapy Access Cap & Canister		
Wall Chart		
T-Handle Allen Wrench		
Casual Water Drain		

Training Date:
Staff:
Supervisor:

Please, make copies of this training checklist for each staff member that will operate the Entree Bath.



Generation II Entree Bath – All Models

OPERATION MANUAL

This page is intentionally left blank