



# Entrée Bath Gen III

## OPERATION MANUAL

### ENTREE BATH TRAINING CHECK LIST

After reviewing the operation and safety instructions in this manual, the Entrée Bath can be operated and all components can be identified

OPERATION:	YES	NO
Position Transfer Chair And Lock The Casters		
Secure Individual In The Transfer Chair		
Transfer Individual Into The Entrée Bath		
Close Tub Door With Push Buttons		
Close Drain With Open/Close Lever		
Adjust Mixing Valve To Desired Temperature		
Fill Reservoir Console And Or Tub		
Use Liquids As Desired		
Use Shower Wand With Shower Wand Control Valve		
Open Drain With Open/Close Lever		
After Water Is Drained, Open Tub Door With Push Buttons		
Transfer Individual Out Of The Entrée Bath		
Return Individual To Room Or Desired Location		
Remove Comfort Pads And Place In Foot Well		
Use Disinfectant To Clean Tub, Transfer Chair And Pads		
Rinse Disinfectant From The Tub, Transfer Chair And Pads		

LOCATE & IDENTIFY:		
Operator Controls And Thermoscopic Mixing Valve		
LCD Thermometer Tub Fill and Reservoir Fill		
Emergency Door Open And GFCI reset		
Liquids Dispensing And Aromatherapy Access Cap & Canister		
Wall Chart		
Casual Water Drain		

Training Date:
Staff:
Supervisor:

Please, make copies of this training checklist for each staff member that will operate the Entrée Bath.